# MULTI TRAY

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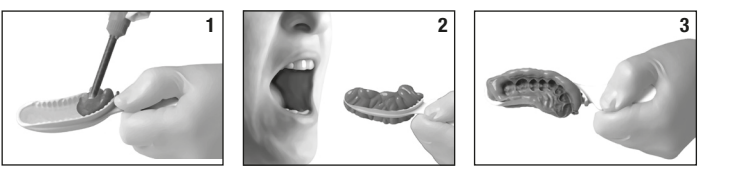
ENGLISH

### Indications/Techniques

- Impressions for fabricating **temporary** crowns/bridges
- Partial anatomical impressions
- One-step precision impressions for fabricating fixed restorations (1–2 unit impression)

### Instructions for Use in Dentistry

- Multi Trays are suitable for use with anatomical impression materials, e.g. Silginat® or also medium- to high-viscosity precision impression materials including in combination with low viscosity materials.
- When using a Multi Tray, check that the distal edge of the tray is lined up to the distal of the last upper tooth.
- Instruct patient to bite on the unloaded impression tray and ensure that the occlusion is correct.
- Load both sides of the tray adequately with the impression material (Figure 1).
- Place the loaded tray into the patient’s mouth within the working time of the impression material. **Ensure that the distal edge of the tray is in the correct position (Figure 2).**
- Ask the patient to bite firmly in the correct occlusion until the material has set fully (Figure 2).
- After the material has set, carefully remove the impression tray from the patient’s mouth (Figure 3).
- Disinfect the impression.
- Store the impression tray until fabrication of the prosthetic restoration is complete.

### Warning

Multi Trays should not be reused. Following intraoral try-in, the impression tray must be used only for the respective patient or disposed of for hygienic reasons.

Multi Trays are unsuitable for use with putty materials.

### Cautions

Do not use after the expiration date.

Multi Trays must be used with firm setting impression materials.

This product does not normally cause an allergic reaction; an allergic reaction to this product cannot, how- ever, be ruled out with sensitive patients.

### Storage Conditions

Store dry and protect against sunlight. Store at room temperature. Avoid extreme temperature ranges.

### Disinfection

Trays with impressions can be disinfected using for example, a 2% acidic glutaraldehyde solution or with a disinfectant solution intended specifically for impression materials. See manufacturer instructions.

### Disposal

Multi Trays and set material can be disposed in the household rubbish.

### Trademarks

Silginat® is registered trademark of Kettenbach GmbH and Co. KG.

### Note

Selected Kettenbach materials are available in certain markets only.

For dental professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the or- der of a dentist.

If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.

### Limitation of Liability

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recom- mendations and information in connection with any use.

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